

TITEMS Diagnostic Emulator

Type 6 Emulator

Interpretation Guide



PASS



PASS: GREEN IS GOOD! Different sterilizers produce varied results depending on quality of steam, as long as the emulator has turned **GREEN** the parameters have been met.

UNEXPOSED



FAIL



FAIL: Any **PURPLE** or **GREY** on the indicator dot is a FAIL. If in doubt do not use.

WET STEAM



WET: Emulator laminate will "bubble" when excessive moisture has been present in the sterilizing process.

SUPERHEATED STEAM



This is a **FAIL**. Emulator will "burn" due to lack of moisture in the steam during sterilization.