



Apply, shape,
smooth,
enjoy!

Direct
Composite
application just
got easier

with GC Modeling Liquid



Since 1921
Towards Century of Health

Simplify your composite work using GC Modeling Liquid with your brush or instruments

With the right tools, a smooth and well-sculpted direct composite restoration is achievable, resulting in a nice aesthetic outcome.

Using GC Modeling Liquid and a brush to adapt composites is a true game changer.



Quick & easy application and shaping

Using a **brush wetted with GC Modeling Liquid** results in a **quick and easy application and shaping** of composites.



A smooth finish for a shorter finishing procedure

Adapting the final layer of composite using a wetted brush aids in achieving a **smoother** surface and it **simplifies the finishing procedure**.



Invisible for great aesthetic results

GC Modeling Liquid becomes **fully transparent after light-curing** — giving **nice aesthetic results**.



The brush technique works well with **GC Modeling Liquid** as a wetting resin.

- **Low viscosity** for ease of use
- **Completely invisible** after light-curing
- **Simple & quick application**
- **350 drops** per bottle
- Compatible with **all direct composites**
- Storage at **room temperature**



Apply, shape, smooth, enjoy!




Dispense one drop of GC Modeling Liquid on a pad.



Protect it from the light with a cover.



Touch the drop with the brush or instrument in order to moisten it.

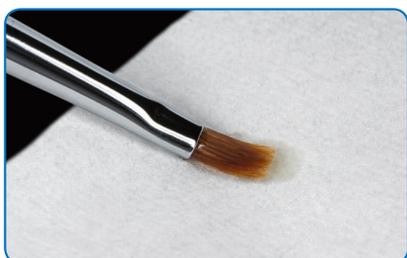

Remove the excess thoroughly using a gauze swab.



Use your instrument or brush to apply the composite.



Use the flat brush to adapt and shape the composite.



Contouring and removal of excess composite using your instrument.



Model the dentin mamelons using a round brush.



Shape and smoothen the final layer of composite using a wetted flat brush or instrument.

Also great for posterior cases

A round brush helps to build the cusps and refine the design of the fissures.



Apply the composite using your wetted instrument.



The brush can be used to adapt the material cusp by cusp.



GC Modeling Liquid and the brush aide to recreate the fissures and refine anatomy.

Get ready for easy composite application

with GC Modeling Liquid & Composite manipulation brushes



GC Modeling Liquid, Refill 6mL

- Composite Manipulation Brush Flat x10
- Composite Manipulation Brush Round x10
- Composite Manipulation Brush handle x2

Discover great handling and aesthetics

with GC paste composites



G-aenial A'CHORD®



G-aenial® Anterior / Posterior



Essentia®



Technique Video



GC Composite Portfolio



Since 1921
Towards Century of Health

GC Australasia Dental Pty Ltd
1753 Botany Road Banksmeadow
NSW 2019 Australia

T: +61 2 9301 8200
E: info.australasia@gc.dental
www.gcaustralasia.com